# C E F A L Y    S O F T W A R E :

## User Manual Manuel d'utilisation

Version 2.0  
Copyright © STX-Med, 2009-2010  
All rights reserved.  
[www.cefaly.com](http://www.cefaly.com)

### Document history

V1.0	First version
V2.0	Support for multiple languages Added manual section in French

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
# User Manual – En

## 1. First time with the Software

Before a first utilisation of the Cefaly Software, the drivers that communicate with the device should be properly coupled with the cable. To do this, one just needs to connect the cable to the USB port (without a Cefaly device) and waits that the cable is ready for use. That is why you will see the following window appearing.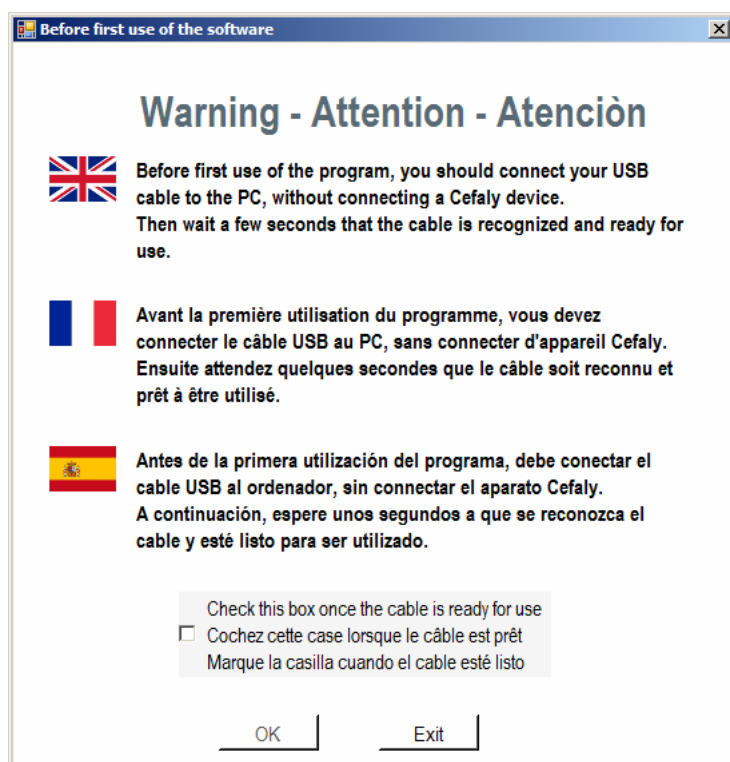

Typically, once your cable is connected, a small pop-up will appear from the notification area (right-bottom part of your screen), stating that the new device is detected, then that it installs it and finally that it is ready for use.


Please note this only happens the first time you connect your cable.

Once that is done, you check the box "Check this box once the cable is ready for use", then click **OK**.

Now you can see the main window. As it is the first time you use it, you can either get data from a device (and thereby create a patient file) or exit.


Before choosing "Get data from a Cefaly device", make sure your cable is connected to the USB port and to a Cefaly device. Then click on the button. After a few seconds the following window appears: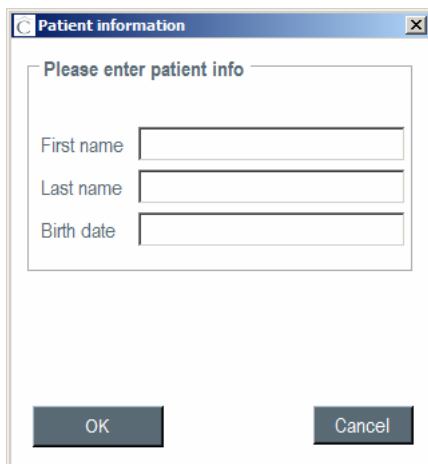

## 2. Standard use

When you start the Cefaly Software, the following window appears.

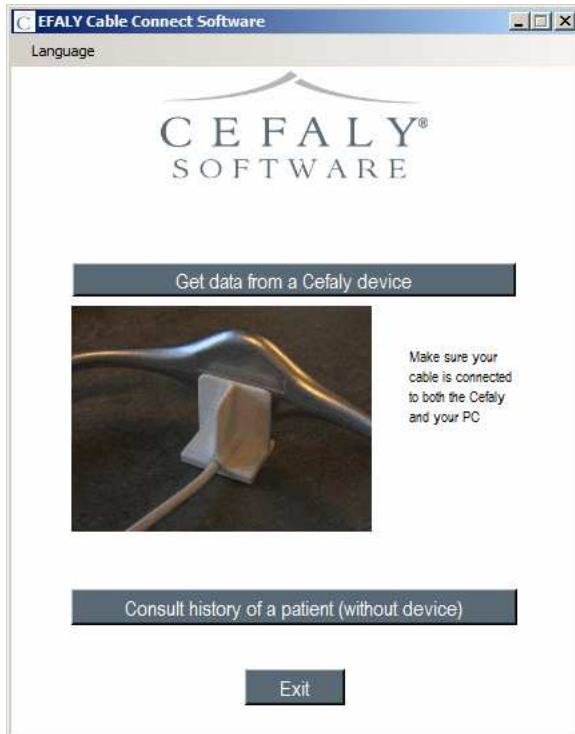It is the main window, representing the different possibilities of the software. You can either collect data from a Cefaly device (new or existing in the database) or consult the history of records for a patient.

### 2.1 Getting data from a Cefaly device

If you connect a Cefaly device that was previously connected and therefore attached to a patient's profile, a window similar to this one will pop-up:

If you want to readdress the device to another patient (and therefore click on the 'Change data file' button), or that the device has never been connected to your PC, the following window will appear:

The screenshot shows a dialog box titled "Patient information". Inside, there is a section titled "Please enter patient info" containing three text input fields labeled "First name", "Last name", and "Birth date". Below this section is a dropdown menu with the text "Or select patient name". At the bottom of the dialog are two buttons: "OK" and "Cancel".

In this form, you can either select an existing patient from the drop down list in the bottom or enter the information (first name, last name and birth date) for a new patient in the part above.

## 2.2 Consulting history of a patient

If you wish to consult some data from a patient, you can get these data by clicking on the button 'Consult history of a patient (without device)'. This will open the following window, where you should select the patient in a drop-down list of available patients.


The screenshot shows a dialog box titled "Patient selection". It contains a single dropdown menu with the text "Please select a patient ...". Below the dropdown are two buttons: "OK" and "Cancel".

### 3. Visualisation of the statistics


This part will explain you the visualisation window and give a brief description of each tab.

There is a menu in the right top corner of the window. Three functionalities are available:

1. You can select another date in the available records of the patient by using the drop-down list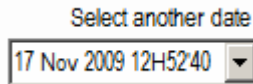

2. You can close the window and return to the main window by clicking 'Return to main'

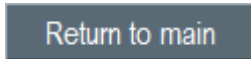3. You can see more info about the software (including the version) by clicking 'About Cefaly Software'

The part in the centre top gives details about the patient (First Name, Last Name, Birth date) together with date indication on the current record (when it was, and how long ago).

**Patient: John Smith 01 01 1900**

---

Data from 17 Nov 2009

1 day(s) ago.

For all tabs except the summary, you can find detailed information on each of the given statistic by clicking on the 'Click here to know more about these statistics' hyperlink below the title.

The statistics are given for the three program of the Cefaly device:

- Program 1, typically indicated in attacks/crisis
- Program 2, typically used for prevention
- Program 3, for well-being/destress

### 3.1 Usage Time tab

In the first tab, the statistics provided are related to the usage time of the device since the last time the statistics were captured. It gives total time and average time per day, and gives a percentage compared to a daily usage of 20 minutes.

**EFALY Cable Connect Software**

**C E F A L Y<sup>®</sup> SOFTWARE**

**Patient:** John Smith 01 01 1900

Select another date  
17 Nov 2009 12:52:40

Data from 17 Nov 2009  
Today

Return to main  
About Cefaly® Software

Usage Time	Intensity	Dose	Sessions	Summary
<b>Usage Time</b>				
<a href="#">Click here to know more about these statistics</a>				
			<b>Program 1 Attacks/Crisis</b>	<b>Program 2 Prevention</b>
				<b>Program 3 Well-being/De stress</b>
<b>Total time</b>	[HH:MM]		00:20	00:06
				00:02
<b>Daily average</b>	[HH:MM]		00:20	00:06
				00:02
<b>% of theoretical</b>	[%]		100	30
				10

Number of days recorded in these statistics : 1

## 3.2 Intensity tab

In the second tab, the statistics provided are related to the intensity of current the patient has received since the last time the statistics were captured. It gives maximum reached and average per session, and gives a percentage compared to the theoretical maximum.

**EFALY Cable Connect Software**

**C E F A L Y**  
SOFTWARE

**Patient:** John Smith 01 01 1900

Data from 17 Nov 2009  
1 day(s) ago.

Select another date  
17 Nov 2009 12:52:40

Return to main  
About Cefaly® Software

**Usage Time** | **Intensity** | **Dose** | **Sessions** | **Summary**

### Intensity

[Click here to know more about these statistics](#)

		Program 1 Attacks/Crisis	Program 2 Prevention	Program 3 Well-being/Destress
<b>Maximum intensity reached</b>	[mA]	16,0	9,0	4,5
→ %	[%]	100	56	32
<b>Average intensity of a session</b>	[mA]	11,4	6,2	3,5
→ %	[%]	100	100	100

1 1

### 3.3 Dose tab

In the third tab, the statistics provided are related to the dose the patient received since the last time the statistics were captured. It gives total applied dose and average dose per day, and gives a percentage compared to a daily usage of 20 minutes at maximum intensity.

**EFALY Cable Connect Software**

**C E F A L Y**  
SOFTWARE

**Patient:** John Smith 01 01 1900

Data from 17 Nov 2009  
1 day(s) ago.

Select another date  
17 Nov 2009 12:52:40

Return to main  
About Cefaly® Software

**Usage Time** | **Intensity** | **Dose** | **Sessions** | **Summary**

### Dose

[Click here to know more about these statistics](#)

		Program 1 Attacks/Crisis	Program 2 Prevention	Program 3 Well-being/Destress
<b>Total Applied Dose</b>	[uC]	0,34	0,00	0,01
→ %	[%]	100	100	100
<b>Daily Dose</b>	[uC]	0,34	0,00	0,01
→ %	[%]	100	16	3

Number of days recorded in these statistics : 1

### 3.4 Sessions tab

In the fourth tab, the statistics provided are related to the sessions of the device since the last time the statistics were captured. It gives the number of sessions completed, repeated and not completed (interrupted or stopped).

**EFALY Cable Connect Software**

**C E F A L Y**  
SOFTWARE

**Patient:** John Smith 01 01 1900

Select another date  
17 Nov 2009 12:52:40

Data from 17 Nov 2009  
1 day(s) ago.

Return to main  
About Cefaly® Software

Usage Time	Intensity	Dose	Sessions	Summary
<b>Sessions</b>				
<a href="#">Click here to know more about these statistics</a>				
			<b>Program 1 Attacks/Crisis</b>	<b>Program 2 Prevention</b>
				<b>Program 3 Well-being/Destress</b>
<b>Completed sessions</b>	[#]	1	0	0
<b>Repeated sessions</b>	[#]	0	0	0
<b>Incomplete sessions</b>	[#]	0	1	1
→ <b>Interrupted</b>	[#]	0	0	1
→ <b>Stopped</b>	[#]	0	1	0

Number of days recorded in these statistics : 1

## 3.5 Summary tab

In the last tab, all available statistics are summarized in a table.

**EFALY Cable Connect Software**

**Patient:** John Smith 01 01 1900

Data from 17 Nov 2009  
1 day(s) ago.

Select another date  
17 Nov 2009 12:52:40

Return to main  
About Cefaly® Software

Usage Time	Intensity	Dose	Sessions	Summary
	<b>Title</b>	<b>Program 1</b>	<b>Program 2</b>	<b>Program 3</b>
	Reset date	17 11 2009	1	
	Total time [HH:MM]	00:20	00:06	00:02
	Daily Average [HH:MM]	00:20	00:06	00:02
	% of theoretical [%]	100	30	10
	Maximum intensity reached [mA]	16,0	9,0	4,5
	% of maximum [%]	100	56	32
	Average intensity of a session [mA]	11,4	6,2	3,5
	% of average [%]	100	100	100
	Total applied dose [uC]	0,34	0,00	0,01
	% of total applied dose [%]	100	100	100
	Daily dose [uC]	0,34	0,00	0,01
	% of daily dose [%]	100	16	3
	Completed sessions [#]	1	0	0
	Repeated sessions [#]	0	0	0
	Incomplete sessions [#]	0	1	1
	Interrupted sessions [#]	0	0	1
	Stopped sessions [#]	0	1	0

## 4. Go further by using Excel

The different records of the statistics are stored in a database that uses Microsoft Excel Document format. This allows you to exploit all the possibilities of software that can read and manipulate .xls files.

### 4.1 Where are my files located ?

In the directory where you installed the Cefaly Software (typically C:\Program Files\Cefaly Software), you will see a 'Clients' subdirectory. In this folder, the different patient files are located. The name of each file is First\_Name Last\_Name Birth\_date as you entered it in the patient information forms.

### 4.2 Important information regarding the data files

As you will see, all files are marked as read-only. The goal is to prevent a misuse or a corruption of the file. **IN CASE YOU MODIFY THE ORIGINAL FILE, THE CEFALY SOFTWARE MIGHT BE UNABLE TO RECOVER AND DISPLAY CORRECTLY THE RECORDS - SO PREFERABLY MAKE A COPY SOMEWHERE ELSE IF YOU WISH TO UPDATE THE FILE.**

### 4.3 Excel file structure

For each patient file, a first tab (worksheet) will contain the information about the patient. The other tabs (worksheets) contain the data of the different records of the statistics. Their name represents the actual date of the record.

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The screenshot shows the 'Sessions' tab in the EFALY Cable Connect Software. The patient is identified as John Smith (01 02 03). The data is from January 22, 2010, 4 days ago. The interface displays a table of session statistics for three programs: Intervention de crise, Prévention, and Relaxation. The statistics are categorized by session status: completed, repeated, incomplete, interrupted, and stopped.

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